

1. Name of the product

Sensory Wagon with Mini Beamer set

2. Product code

21964010

3. Colour

See image

4. Brief description

For safety and service reasons, this Snoezelkar with Mini Beamer set is delivered in a wooden transport case.

The projector shown is replaced by a Mini Beamer set.



*We recommend keeping the wooden transport box for possible service purposes.

1. Open the transport box with a Phillips screwdriver.
2. Release the brakes on the front wheels at the bottom of the Snoezelen Wagon and drive the car out of the transport box.
3. Hang the fibre optics next to the cart.
4. You can see the key that opens the front and back of the Snoezelen Wagon. The key of the Snoezelen Wagon is in the handle of the low door, it is attached with a tie wrap.
5. Open the low door at the front of the Snoezelen wagon with the key provided.
Take the beamer, speakers and Laser Stars Projector out of the cabinet and place them on the cart.
6. Connect the projector, beamer and speakers to their respective products at the top of the cart, the small cable to the grey Laser Stars projector and the thicker cable to the beamer.
7. Then connect the speakers to the beamer using the cable supplied. The connections speak for themselves.
8. Take the 3-metre-long power cable out of the cabinet and place one side of this power cable in the socket on the outside of the Snoezelen Wagon (right side as seen from the front) and place the other side of the power cable in the socket in the room where the Snoezelen Wagon will be used.
9. Then take the Adaptor (connection for the bubble tube) out of the cabinet and place it in the other connection which can be found on the outside of the Snoezelen Wagon under the fibre optics.
10. Use the supplied key to open the rear side of the Snoezelen Wagon. Here you will find a removable part with the bubble tube and mirror at the back. This wooden "panel" can be easily removed by gently pulling it backwards by the handle.
11. Place the wheeled section with the bubble unit next to the Snoezelen Wagon and block the wheels of the wheeled section before use.
12. Fill the bubble tube for 80% of water, leaving a bit of room for the bubbles.
13. Pierce the other side of the small power cable, which has just been placed on the outside of the Snoezelen Wagon, into the bubble tube, so that it too is supplied with power.
14. Open the cabinet at the front of the Snoezelen Wagon again; in this low cabinet you will find a switch box with 6 red buttons. These buttons are the on/off buttons for the electronic products and components that are used in and on the Snoezelen Wagon. The diagram below shows which button is for which device (from left to right).



Please pay attention: Always plug the Snoezelen Wagon on the outside first and only then switch on all the red buttons!

Button 1 Bubble tube

Button 2 Fibric optics

Button 3 Aquarium

Button 4 Laser Stars Projector

Button 5 Mini-beamer + speakers

Button 6 Aroma diffuser

15. Don't forget the cardboard box that comes with it! This can be found in the lower cabinet. This box contains the 4 aroma mixtures (these fragrances can be used with the aroma diffuser).
16. Put 4 drops of one of the scent mixtures on the aroma pad at the bottom of the aromatherapy spreader. Unscrew the bottom of the aromatherapy diffuser, put 4 drops of a fragrance mixture of your choice on the aroma pad and close the bottom of the diffuser.
17. The Snoezelen Wagon is now ready to use!

5. Sense Stimulations

Feel, Smell, Hear and See

6. Specifications

Size: 65 x 60 x 147 cm

Power supplies: 220 V

7. Included

1 x Mini Beamer set

1 x Speaker set

1 x Bubble tube

1 x Fibre optics

1 x Aquarium

1 x Laser Stars Projector

1 x Light Source for fibre optics

4 x Essential oil

1 x Aroma diffuser

For more information you can visit our website www.nenko.com.

